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Z NWW M DS V2 0716

Dear Customer,

We are delighted that you have chosen our **night guard sleeping vest**.

If you have any questions, contact the customer service department via our website:

www.service-shopping.de

Intended Use

- This article is intended to prevent the person wearing it from lying on their back while sleeping.
- The article is **not** suitable for people with allergies to polyamide, polyester or elastane.
- The article is **not** suitable for medical or therapeutic purposes. Consult your general practitioner if you have any sleep problems or back problems.
- Do not use the article
 - if you suffer from spinal complaints
 - if you are convalescing after surgery on a slipped disc
 - if you have positional vertigo
 - if you have sleep apnoea syndrome and arousals that are associated with this.
- Pregnant women should not use the article or use it only after consultation with their doctor.
- If you have any health concerns, consult your doctor before using the article.
- The article is for personal use only and is not intended for commercial applications. Use the article only as described in the instructions. Any other use is deemed to be improper.
- The following are excluded from the warranty: all defects caused by improper handling, damage or unauthorised attempts at repair. This also applies to normal wear and tear.

Safety and Care Instructions

- There is a danger of suffocation! Keep the packaging material away from children and animals.
- The article is **not** a toy! Do not leave it unattended in the vicinity of children and animals.
- If you have allergies to polyamide, polyester or elastane, the article may cause skin rashes and skin irritations. Do not use the article if you notice any skin rashes or skin irritations. In this case, consult a doctor immediately.
- If you notice any complaints when wearing the article, stop using the article. If the complaints persist, consult a doctor.
- Due to the manufacturing process, storage and/or dispatch, you may notice a slight odour the first time you unpack the article. This odour will disappear after a little while and is harmless. If the odour bothers you, clean the article a few times in the manner described in the "Care" chapter.
- Keep the article away from sharp objects.
- Maintain a sufficient distance from heat sources (e.g. ovens, fireplaces, candles etc.).
- Do not stand on the article. Do not overload the article by forcibly compressing it, bending it or pushing it in.

Application

- Put the article on like a conventional vest and press down the Velcro fastener on the front.
- When you put on the article, make sure it is not too tight against your body. The vest should have slight room for manoeuvre so that it can move back and forth slightly while you are sleeping.

The body will slowly get used to this form of sleeping. This is why at first we recommend that you do not wear the vest every night.

Materials

Protector: 100% PU cold foam

Protector cover: 85% polyamide, 15% elastane

Vest: 90% polyester, 10 % elastane

Belt: 60% polyamide, 40% elastane

Velcro fastener: 100% polyamide

Care

To clean the article, open the zip on the back and remove the protector.

Caution: Only the vest is washable, the protector must **not** be washed. If necessary, allow the protector to thoroughly air or wipe it with a slightly damp cloth.

Hand wash up to max. 30°C

Do not bleach

Do not dry in a tumble dryer

Do not iron hot

Normal dry cleaning

- Before washing, close the zipper and the Velcro fastener in order to avoid material damages.
- Wash with a mild detergent only.
- Do not use fabric softener.
- Do not wring out.

Versions

00716 - Size XL/XXL

00721 - Size M/L

Disposal

The packaging material is recyclable. Please dispose of it in an environmentally friendly manner and take it to a recycling centre.

Dispose of the article in an environmentally friendly manner. You can obtain more information from your local authorities.

Customer Service / Importer:

DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Germany • ☎ +49 38851 314650 *)

*) Calls to German landlines are subject to charges. The cost varies depending on the service provider.

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All rights reserved.
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Duitsland • ☎ +49 38851 314650 *)