# **HYBERNITE**

**Superday**


## **Instructions For Use**

**Hybernite ROC 2300 Rainout Control System**

## Instructions For Use

*The Hybernite Rainout Control (ROC) System condensation control system is a tube system for adult patients who require respiratory support via positive pressure ventilation. The system consists of a heated therapy tube and an external power supply unit. The heating prevents condensation in the therapy tube. The Hybernite condensation control system can be used in sleep laboratories and at home.*

### Hybernite Rainout Control System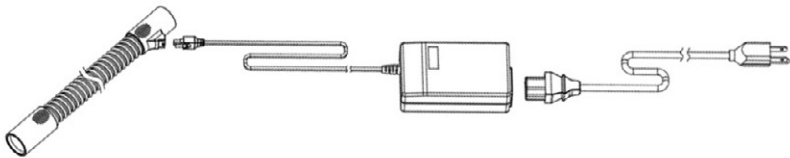

### Table of Contents

1. Warnings and Cautions
2. Intended Use
3. Device Description
4. Directions for use
5. Cleaning
6. Operating Conditions
7. Storage and Transport Conditions
8. Frequently Asked Questions
9. Product Specifications
10. Replacement Components
11. Warranty
12. Disposal
13. Symbols on the Packaging

**PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE USE**

# 1. Warnings and Cautions

## Usage:

- Do not cover the Heated Breathing Tube with any object including cushions, blankets, a textile insulation sleeve, a plastic sleeve, etc, to avoid an elevated airflow temperature.
- Always use the Hybernite ROC System in combination with humidification to avoid an elevated airflow temperature.
- Always ensure that a positive airflow is established through the Hybernite Heated Breathing Tube prior to connecting the power to the tube, to avoid an elevated air temperature within the tube.
- Do not operate your Hybernite ROC System close to other heat sources including lamps, electric blankets, etc., to avoid an elevated airflow temperature.
- Do not use the Hybernite ROC System when the room temperature exceeds 30°C (or 86°F) in order to avoid an elevated airflow temperature.
- Always connect-disconnect the breathing tube from the humidifier and mask by using the grips on the cuffs, in order to avoid damage to the breathing tube
- Do not pull the tube when disengaging the tube from the humidifier.

## Cleaning:

- Do not heat or sterilize the Hybernite Heated Breathing Tube.
- Do not use harsh soaps, aggressive detergents, solvents or alcohols to clean the breathing tube.
- Do not use a dishwasher to clean the Heated Breathing Tube.
- Do not use brushes or objects to clean the inside of the Heated Breathing Tube as it may damage the tubing.
- Do not use (microwave) ovens, fans or any other appliances to dry the Heated Breathing Tube after washing. See cleaning instructions.
- Only reassemble the system when the tube is fully dry.
- Do not clean the power supply using water or any other liquid.
- Do not place or drop the Hybernite power supply into water or other liquid.
- If the Hybernite power supply does fall in water, do not attempt to retrieve it. Unplug the power supply immediately.

## General Handling

- Do not use the heated breathing tube longer than 12 months under any circumstances.
- Replace the breathing tube immediately when damaged, and as often as recommended by your physician or respiratory therapist.
- If the breathing tube is damaged (having holes, kinks, tears, exposed heating wires, etc.) or not functional, do not try to repair the breathing tube yourself. Immediately replace your heated tube.
- Immediately replace your power supply when the power cord, lead or connectors are damaged.
- Only use replacement parts (Heated Breathing Tube, power supply) recommended by Plastiflex Healthcare for your Hybernite Rainout Control System.
- Only use the Hybernite Heated Breathing Tube with the included power supply. Do not use with any other power source.
- Do not disassemble or intentionally damage the Heated Breathing Tube.
- Do not insert or drop any object in the opening of the Heated Breathing Tube.
- Do not place materials or objects on the tube as it can damage the Heated Breathing Tube.
- Dispose of the tube correctly according to your local government regulation.
- Upon permanent disposal, mark the tube so that it is not used by any other patient.

## Power Supply:

- Do not use the Hybernite V12 Power supply in an oxygen enriched environment.
- Do not use the Hybernite V12 Power supply in an area with anesthetic gases.
- Disconnect the Hybernite V12 Power supply in case of problems, maintenance or cleaning.
- Do not use the Hybernite V12 Power supply at height higher than 2000m above sea level.

## General:

- US Federal law restricts this device to sale by or on the order of a physician
- Retain these instructions for use for future reference.
- When using electrical products, always follow usual safety precautions.
- The Hybernite Heated Breathing Tube is for single patient use only.

## 2. Intended Use

The Hybernite Rainout Control System is used to prevent condensation of the therapy air in the tube. It is used by patients receiving respiratory support via positive pressure ventilation.


The Hybernite Rainout Control System is intended for non-invasive respiratory therapy in the home or sleep laboratory, and can also be used with an additional oxygen supply.

The Hybernite Rainout Control System is intended to help patients experiencing problems with condensation in the tube due to the use of a humidifier. The Hybernite Rainout Control System prevents condensation and thereby contributes to greater comfort during therapy.


The Hybernite Rainout Control System may only be used by adult patients.

## 3. Description of the Hybernite Rainout Control System

Hybernite HBT-300 Heated Breathing Tube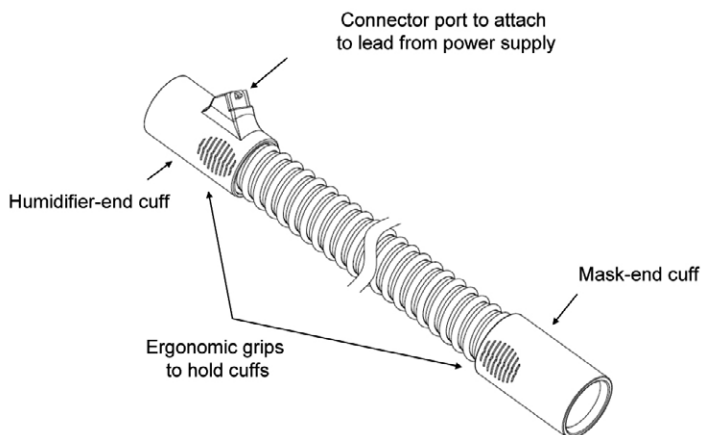

Hybernite V12 Power supply Unit with power cable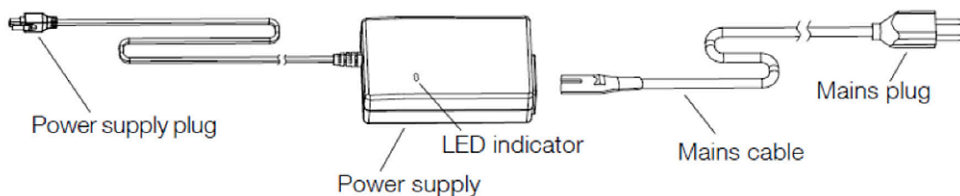


## 4. User's Manual

Humidification System compatibility

The Hybernite Rainout Control System consists of a heated tube and an independent power supply. The Hybernite Heated Breathing Tube has standard cuffs on both the humidifier-end cuff and mask-end cuff. As such, the Hybernite Heated Breathing Tube can be connected to heated humidifiers that have standard male outlet connectors.

## How is it connected?


1. Remove the packaging from the therapy tube.
2. Before installation, check the tube, socket and power supply unit for damage.
3. Insert the power plug into the socket on the humidifier cuff.
4. Connect the humidifier cuff with the humidifier's air outlet.
5. Connect the mask cuff to the mask.
6. Turn on the humidifier and ensure there is a flow of air running through the breathing tube.
7. Plug the power plug into a power socket and place the power supply unit onto the floor.
8. Use the LED indicator on the power supply to check if the power supply unit is working. When working properly, the LED lights up green.
9. Put the mask on and start respiratory therapy according to your physician or respiratory therapist's instructions, as well as the respiratory therapy device's user's manual.
10. Ensure that the breathing tube is not covered by objects, cushions or blankets during use.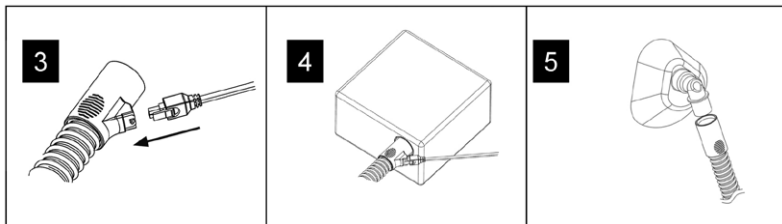

## Disconnecting the power supply plug from the humidifier cuff

Press down the lever (1) at the rear and carefully remove the plug (2) from the humidifier cuff.

Always disconnect the power to the Heated Breathing Tube prior to switching off the humidifier and air flow.

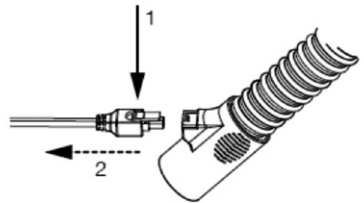## 5. Cleaning

### Warnings:

1. Do not clean the power supply with water or any liquid.
2. Do not use harsh soaps, detergents, solvents or alcohols to clean the tube.
3. Only reassemble the system when the tube is fully dry.
4. Do not use a dishwasher to clean the Heated Breathing Tube.
5. Do not use the Hybernite System, if the room temperature higher than 30°C. Increased therapy air temperature can cause irritation of the airways.

Clean the tube daily. To do this, proceed as follows:

1. Remove the power supply plug from the humidifier cuff.
2. Clean the tube in lukewarm, mild soapy water.
3. Rinse out the tube with clean water.
4. Gently tap on the humidifier cuff to remove the remaining water from the plug connection.
5. Hang up the tube with both cuffs pointing downwards.
6. Let the tube dry completely.

## 6. Operating Conditions

Operating Temperature: 5°C – 30°C (41°F – 86°F)  
Relative humidity: 5% – 95%

## 7. Storage and Transport Conditions

Storage temperature - 20 °C (-4°F) ... + 60 °C (120°F)  
Relative humidity 5 % ... 95 %  
Storage conditions The Hybernite System should be transported and stored in the original packaging and protected from dust and exposure to direct sunlight.

## 8. Frequently Asked Questions

*Question:* What happens if I connect the mask-end cuff to the humidifier and the humidifier-end cuff to the mask?

*Answer:* The Hybernite System works exactly as if the tube was connected correctly (mask cuff to mask and humidifier cuff to humidifier). However, for safety reasons and to ensure maximum comfort, connect the tube as described in the user's manual.

*Question:* Can I use the tube without a power supply?

*Answer:* Yes, the tube will then works like a standard tube for respiratory therapy devices. Respiratory air condensation can however form in the tube.

*Question:* When do I have to replace the tube?

*Answer:* The tube must not be used for longer than 12 months. If the tube is damaged or a leak or malfunction occurs, then the tube should be replaced immediately.

*Question:* Which humidifier or flow generator can I use the Hybernite System with?

*Answer:* The cuffs of the Hybernite have the same diameter as standard tubes for respiratory therapy systems. Therefore, you can use the tube with all humidifiers that are intended for the connection of a standard tube: examples: ResMed HumidAire 2i; ResMed HumidAire 3i; ResMed H4i; ResMed H5i; Philips Respironics System One humidifier; Respironics M-series humidifier; Respironics Legacy Series humidifiers; Breas HA20 Humidifier; Devilbiss Heated Humidification systems; Weinmann SOMNOcomfort 2; Weinmann SOMNOsoft 2; Weinmann SOMNObalance; Hoffrichter Point2; Sefam Dreamstar; Sefam Sandman; Apex; BMC; etc

*Question:* How do I get maximum comfort from using the Hybernite Rainout Control System?

*Answer:* The maximum comfort will be different for every patient. The patent is advised to change the settings of the humidifier to find the zone of optimal comfort. It can be advised to increase the settings of the humidifier to the mid or top third of the range (e.g. setting 3, 4 or 5 if the range is from 1 to 5 or settings 4, 5 or 6 if the range is from 1 to 6).

## 9. Product Specifications

Hybernite HBT 300 Heated Breathing Tubing  
Length 1.83 m  
Internal Diameter 19 mm  
Flow resistance  $60 \text{ l} \cdot \text{min}^{-1} \leq 0.1 \text{ kPa}$   
Tubing Compliance Standards ISO 5367:2000  
ISO 8185:2007  
Water resistance IPx3  
Electrical Safety Standard EN IEC 60601-1  
Type B Applied Part 

Hybernite V12 Power supply:	
Supply Frequency	47-63Hz
Supply Voltage to power supply	100V - 240V
Supply Voltage and current to tube	12 Volt (max. 1,25 A)
Power Heated Tube	18W max
Electrical Safety standard	EN IEC 60601-1
Electrical Safety Class IEC 60950	Class II
Water resistance power supply	IPx1

## 10. Spare Tube/Parts

Hybernite HBT-300 Heated Breathing Tube

The Hybernite HBT should not be used for longer than 12 months and should be replaced once this period is over. The Hybernite HBT are intended exclusively for use with the Hybernite Rainout Control System. Should problems occur, ask your physician, respiratory therapist or nursing staff for support in replacing components.

## 11. Warranty

- The Hybernite® HBT warranty is 30 days under normal operating conditions. This is based on 8 hours/day usage.
- The Hybernite® V12 PSU warranty is 2 year at continuous 24 hours per day usage. (At 100- 240 VAC, 47-60Hz., and rated load current.)

Warranty and guarantee expires, if:

- interventions, modifications, extensions, calibration, repairs and maintenance are carried out by persons who are not authorized by the manufacturer,
- other manufacturers' accessories and spare parts are used that have not been approved by the manufacturer for use the product,
- the product is used other than as described in the user's manual,
- wearing parts are worn,
- product damages are caused by force majeure (e.g. by lightning) or
- the hygiene and cleaning instructions described in the user's manual have not been complied with.

Statutory warranty rights remain unaffected by this.

## 12. Disposal

The Hybernite tube and the power supply unit must not be disposed of with the household waste. They must be disposed of at a communal collection point or public waste disposal authority. The packaging can be disposed of with the household waste.

## 13. Symbols on the Packaging

 Observe the warning and safety instructions in the user's manual.

 Follow the user's manual

 Do not use the product, if the package is damaged

 Manufacturer

 Date of manufacture

 Article number

 Batch designation

 Manufacturer's CE mark

**Rx Only** Caution: law prohibits dispensing without prescription. Only applicable for US residents

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Copyright Plastiflex Group nv – 2009 2010.